

Mascarilla Quirúrgica Desechable Tipo IIR

Disposable Surgical Mask Type IIR

CÓMO COLOCARSE LA MASCARILLA CORRECTAMENTE
HOW TO WEAR THE MASK CORRECTLY

- 1) Antes de usar la mascarilla, lávese y desinfecte sus manos.
Wash your hands with sanitizer before wearing a mask.



- 2) Identificar la parte superior de la mascarilla (posición pinza nasal)
Find the upper part of the mask (with the nose clip)

- 3) Colóquese los arneses tras las orejas.
Place the straps over your ears.



- 4) Ajuste el clip metálico para ajustarlo a la forma de su nariz.
Adjust the metal clip to fit the shape of your nose.



- 5) Estire la mascarilla para cubrir su barbilla.
Stretch the mask to cover your chin.



- 6) Tras finalizar, no tocar la mascarilla con las manos.
Once completed, do not touch the mask with your hands.



Ningbo Green Textile Co., Ltd
No. 498, 3rd Xingci Road, Hangzhou Bay New Zone,
Ningbo, Zhejiang, China.



CMC Medical Devices & Drugs S.L.
C/ Horacio Lengo Nº 18 CP 29006, Málaga-Spain
info@cmcmmedicaldevices.com

BFE \geq 98%



20201016



10.16.2020



10.2022



Not be used twice



Prevent exposure to the sun



NON STERILE




EN14683:2019 + AC: 2019
TYPE IIR

Cintas elásticas para ajuste tras las orejas
Elastic band to fit over ears

Clip de ajuste nasal integrado
Integrated nose clip

Eficacia de filtración bacteriana \geq 98%
Bacterial filtration efficiency \geq 98%

Triple capa
3 Layer construction

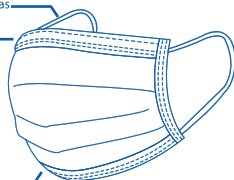

USO DEL PRODUCTO

Esta mascarilla quirúrgica está destinada al uso por personal médico durante operaciones no invasivas para cubrir boca, nariz y barbilla del usuario para proteger directamente, a través de una barrera física, contra microorganismos patógenos, fluido corporal y partículas, etc.

PRODUCT USE

This medical mask is intended for use by medical personnel during non-invasive operations to cover the wearer's mouth, nose and chin to directly protect against the pathogenic microorganisms, body fluid and particles, etc. through a physical barrier.

Composición:

Capa 1: Tejido No Tejido Polipropileno
Capa 2: Tejido No Tejido Soplado (meltblown)
Capa 3: Tejido No Tejido Polipropileno

Composition:

Layer 1: PP non-woven
Layer 2: Meltblown
Layer 3: PP non-woven

10 Pcs

17.5 x 9.5 cm

Item no.: Green2020-32